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## L'app SwissCovid: Problemi noti

Data:

10 settembre 2020

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GitHub Issue #290: No successful sync on iOS 13.7 and iOS 14 beta [11]

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GitHub Issue #187: App is not fully compatible with iOS 14 [3]

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«*ExposureNotification isn't currently available in iOS 14 beta. Use iOS 13.6 beta to preview the latest functionality. (64433241)*»

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GitHub Issue #100: Synchronising report error (AGAEN39508) [6]

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GitHub Issue #133: An unexpected error occurred when synchronising the notifications (AGAEN10.-2) [7]

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GitHub Issue #169: 0 keys displayed in "Exposure checks" menu [5]

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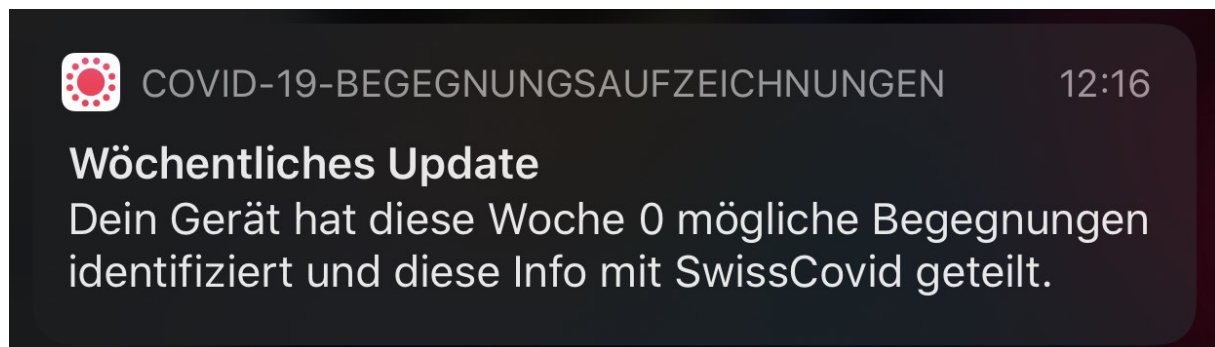
GitHub Issue #228: iOS 13.6: Misleading German translation of "Weekly Update" notification [2]

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### iOS: Popup del sistema «Region Changed»

GitHub Issue #127: Geofencing active: Exposure Notification Region Changed [2]:

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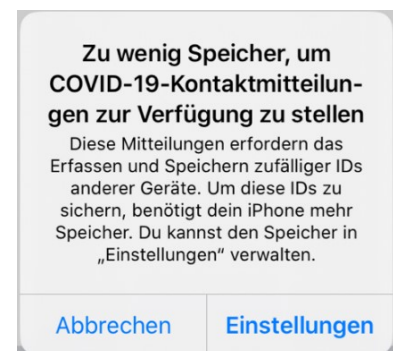
GitHub Issue #162: IEN11 and Insufficient Memory popup [3]:

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
GitHub Issue #163: iOS 13.5 Background Task not always triggered consistently [4]

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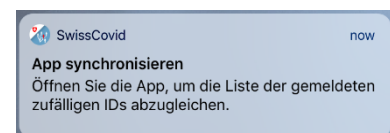## Sorgenti

- [1] SwissCovid codice sorgente: <https://github.com/DP-3T/>
- [2] iOS 13.6: Misleading German translation of "Weekly Update" notification #228 <https://github.com/DP-3T/dp3t-app-ios-ch/issues/228>
- [3] App is not fully compatible with iOS 14 #187 <https://github.com/DP-3T/dp3t-app-ios-ch/issues/187>
- [4] Release notes iOS 14 Beta 2: <https://developer.apple.com/documentation/ios-ipados-release-notes/ios-ipados-14-beta-release-notes>
- [5] 0 keys displayed in "Exposure checks" menu #169 <https://github.com/DP-3T/dp3t-app-android-ch/issues/169>
- [6] Synchronising report error (AGAEN39508) #100 <https://github.com/DP-3T/dp3t-app-android-ch/issues/100>
- [7] An unexpected error occurred when synchronising the notifications (AGAEN10.-2) #133 <https://github.com/DP-3T/dp3t-app-android-ch/issues/133>
- [8] Geofencing active: Exposure Notification Region Changed #127 <https://github.com/DP-3T/dp3t-app-ios-ch/issues/127>
- [9] IEN11 and Insufficient Memory popup #162 <https://github.com/DP-3T/dp3t-app-ios-ch/issues/162>
- [10] iOS 13.5 Background Task not always triggered consistently #163 <https://github.com/DP-3T/dp3t-app-ios-ch/issues/163>
- [11] No successful sync on iOS 13.7 and iOS 14 beta #290 <https://github.com/DP-3T/dp3t-app-ios-ch/issues/290>